

# KATANA™ Cleaner

For intra- and extra-oral use



## For all restorative procedures

Removes contamination to optimise your adhesive procedures. During trial fitting your restoration might become contaminated with proteins reducing the bond strength. KATANA™ Cleaner has a high cleaning effect due to the surface active characteristic of MDP Salt.

In contrast to other cleaners, KATANA™ Cleaner has a pH value of 4.5<sup>1</sup> which allows usage not only extra-orally but also intra-orally. It is the easy way to optimise your bondings including cementation procedures.

1. Measuring method: JIS Z 8802:2011

## Wide indication range: intra- and extra-oral

Due to its mild pH value of 4.5, KATANA™ Cleaner can be used not only extra-orally but also intra-orally, making it an all-purpose universal cleaner for both - tooth structure and prosthetics.



### Intra-oral:

Cavity, Abutment, Root Canal



### Prosthetics:

Ceramics (Zirconia, Lithium Disilicate, Dental Porcelain)

Resin Based Materials (CAD/CAM Crowns, Composite Resin)

Metals (Precious/Non Precious Metal Alloys)


Posts (Glass Fiber Posts, Metal Posts)




## Easy & Fast Procedure

**Rub in, rinse and dry.** That's all you need to do to optimise your adhesive procedure.

Unlike other cleaners that require longer rubbing time, and are limited to extra-oral use only, **KATANA™ Cleaner only requires 10 seconds and can be used both intra- and extra-orally.**

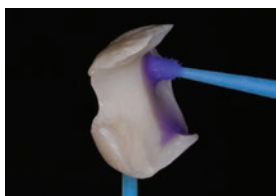

Apply and rub KATANA™ Cleaner for at least 10 seconds. Both to the prosthetic and the intra-oral substrate.

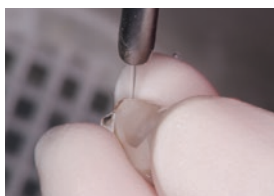

Rinse and dry.



Cement the restoration according to the manufacturer instructions.

Pictures courtesy  
of Dr. Shoji Kato

## Cleaners Comparison\*

	KATANA™ CLEANER	IVOCLEAN** (IVOCLAR VIVADENT)	ZIRCLEAN** (BISCO)
Intra-oral Use on Tooth Structure	Yes	No	No
Intra-oral Use on Implant Abutments	Yes	No	No
Extra-oral Use	Yes	Yes	Yes
Active Ingredient	MDP Salt (Surfactant)	Sodium Hydroxide	Potassium Hydroxide
pH	4.5	13-13.5	13
Application Time	10 Seconds	20 Seconds	20 Seconds
Handling	No Shaking	Shake Before Use	No Shaking

\* According to the Instructions for Use and official documentation

\*\* Not a trademark of Kuraray Noritake Dental Inc.

## Product assortment

ARTICLE NR	PRODUCT DESCRIPTION	CONTENTS
#3970-EU	KATANA™ Cleaner	4.0 ml

SCAN TO LEARN MORE

